


HOW TO ADD A QRS BOOKMARK TO INTELEVIEWER


1. Open IntelViewer on your PC- ensure you have applied and been approved for a QRS PACS user account (every imaging provider has its own PACS, usernames and passwords given out by other providers will not work)
2. Hit "Edit Bookmarks"




3. Hit "Add"

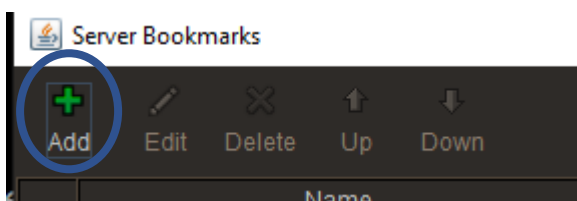


4. Under Server URL, type in or copy and paste this text-
<https://pacs.qldradiologyspecialists.com.au>

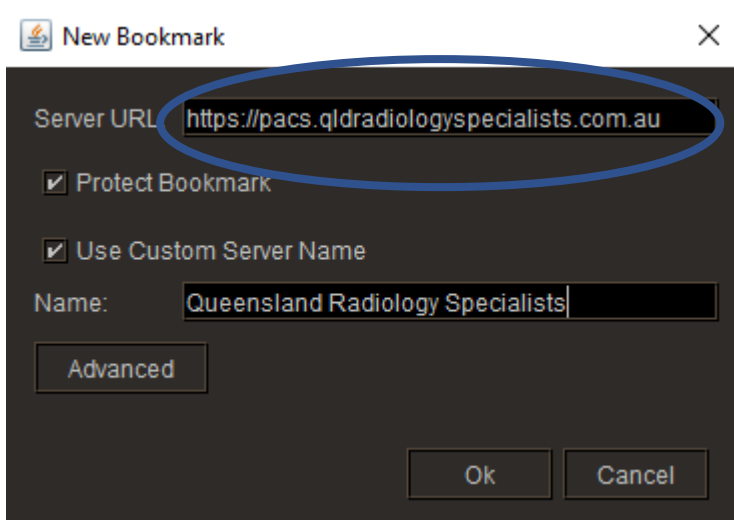


5. Tick "Protect Bookmark"

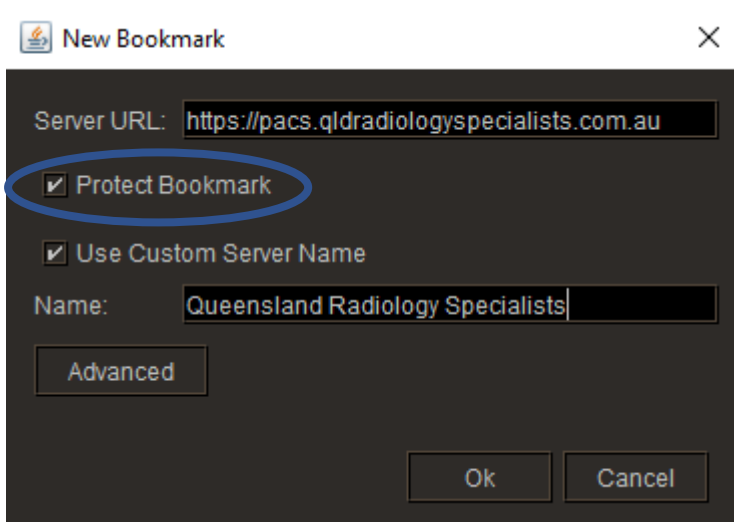

6. Tick "Use custom server name" and type in "Queensland Radiology Specialists"

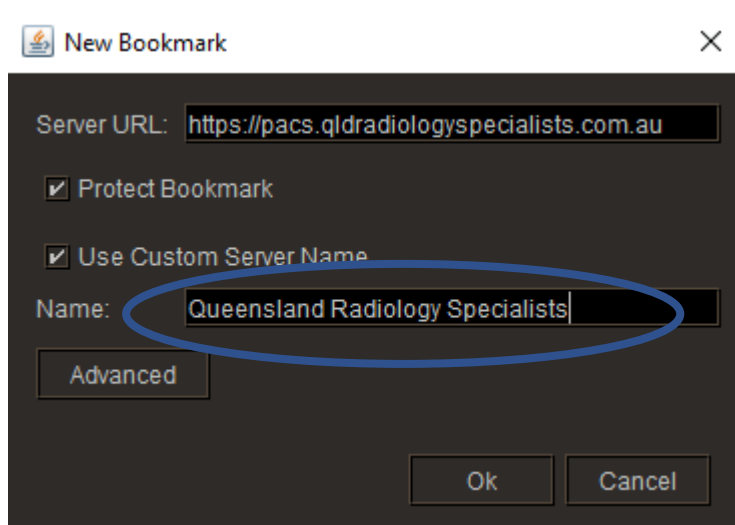

7. Hit "Ok"

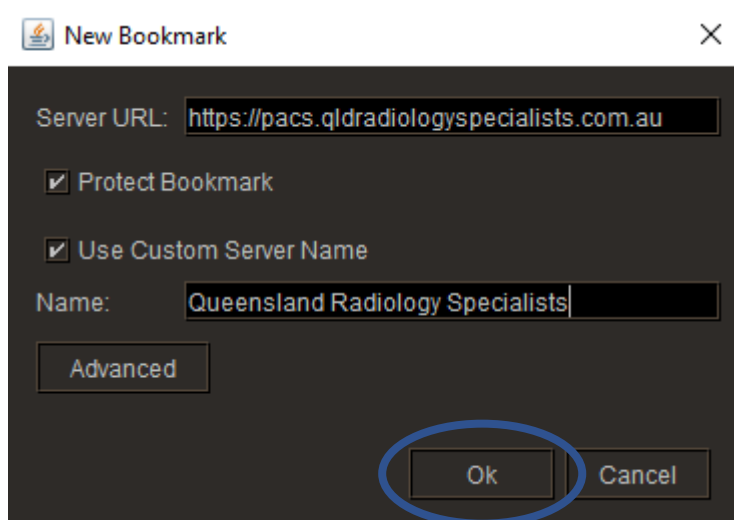

8. Select "QLD Radiology Specialists" on the left of the home screen



9. Type in your username and password you were given when you applied for your account (usually your first initial and your full lastname eg. Dr John Smith would be jsmith). Email info@q-rad.com.au or call 0455 191 067 for assistance.